



TEAM VISTA™ BY HU-FRIEDY

The Complete Waterline Security System



Team Vista provides a more effective and economical way to keep waterlines safe and clean. This easy-to-use solution is the **only system that provides both an organic irrigant and antimicrobial cleaner** for enhanced patient care.

- For use in independent water bottle systems
- Provides continuous cleaning of waterlines for up to one week
- Non-corrosive - gentle and safe on equipment
- One year supply per operator

VistaTab™


Dental Waterline Cleaner Tablets

VistaTab is an antimicrobial tablet used for the cleaning and control of microbial contaminants in dental unit waterlines. When placed in water, the solid tablet quickly dissolves to create a non-corrosive solution that is **seven times more effective than bleach**.

VistaClean™

Irrigant Solution Concentrate

VistaClean is a daily irrigant solution derived from organic citrus botanicals and is safe for use with patients. It helps protect waterlines and equipment from deposits and scale and works to emulsify organic and inorganic contaminants that can build up in waterlines. When used as directed VistaClean leaves no taste or odor and is **completely non-toxic, environmentally friendly**, non-corrosive, and has no negative effect on bond strength.

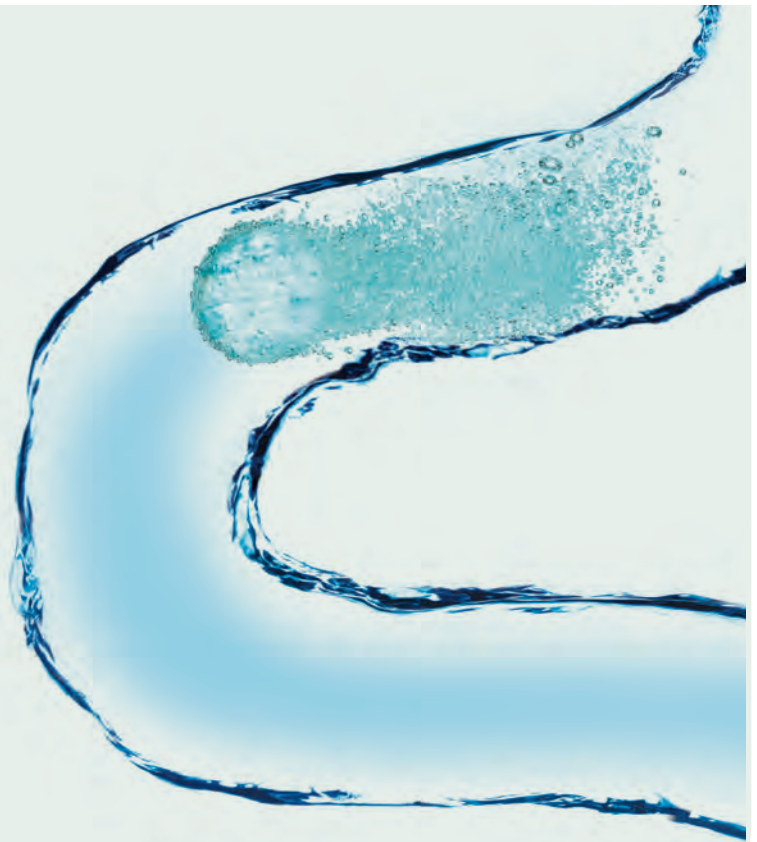

How the best perform



“ It is important to properly treat your dental unit waterlines as a component of a total infection control program. Team Vista is an easy, effective system to both periodically clean and maintain dental unit waterlines. ”

-John Molinari, PhD

THE CDC RECOMMENDS...

that dental unit waterlines be tested on a regular basis to maintain its established standard of less than 500 colony-forming units per milliliter.*

If dental unit waterlines are not maintained on a regular basis, bacteria will develop on the inner surfaces of the tubing, blocks and other system components.

Simply maintaining the waterlines is not enough; it is a two part process of cleaning and maintenance. Over time, bacteria can overwhelm the waterline environment and begin to create microbial deposits. Using a daily irrigant, like VistaClean, and cleaning the waterlines regularly with an antimicrobial, like VistaTab, will clean and control microbial contaminants to provide improved water quality while helping to protect patients, staff, and equipment.

* Reference: Centers for Disease Control and Prevention. Guidelines for Infection Control in Dental Health-Care Settings - 2003. MMWR 2003; 52 [No. RR-17]: [p.31].

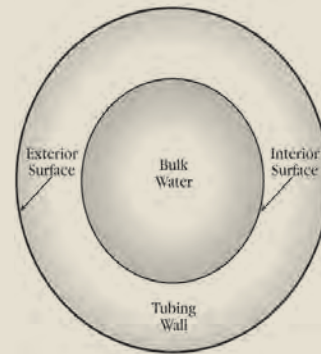


Figure 1. Clean dental waterline tubing. No biofilms or free-floating organisms present.

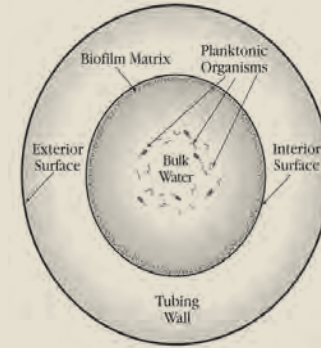


Figure 2. Contaminated dental waterline. Biofilm on interior surface with abundant free-floating organisms.



The following **TEAM VISTA SOLUTIONS** are now available in Canada:



**Team Vista
Dental Waterline Cleaner**
IMS-1450CA (Canadian Part-code)
This complete system contains:
1 - One Ounce Bottle of VistaClean Irrigant Solution
15 - VistaTab Cleaner Tablets
(One-year supply per operator)



**VistaTab
Dental Waterline
Cleaner Tablets**
IMS-1451CA (Canadian Part-code)
This box contains: 15 Tablets (1.7 grams)
Net Weight 22.5 grams (0.90 oz.)



**Vista Clean
Irrigant Solution
Concentrate**
IMS-1452
One Fluid Ounce Bottle (29.57 ml)

Conversions: 500 ml = 16.91 oz. 750 ml = 25.36 oz.
1.7 grams = 0.06 oz. 28.35 grams = 1 oz.

Visit us online at www.Hu-Friedy.com

Call 1-800-Hu-Friedy or contact your authorized Hu-Friedy representative for more information.
©2011 Hu-Friedy Mfg. Co., LLC. All rights reserved. Hu-Friedy and the Hu-Friedy oval logo are registered trademarks of Hu-Friedy Mfg. Co., LLC.
Team Vista, VistaTab, and VistaClean are all trademarks of Vista Research Group, LLC. IMS-337/0411

